



Gasket

- Check to make sure gasket is in bottle cap.
- The gasket has a ridge on the outer edge of one side. The other side of gasket is flat.
- The flat side should be against bottle cap and ridge side should face outward to seal around bottle.

Trouble Shooting

Breast cups not collapsing and inflating

Tighten bottles

(use PJ's Comfort bottles if you are using another bottle type and tightening bottle does not work)

Double check to see if **gaskets are in bottle caps**

Double check to see if the **gasket is lying flat and the ridge on the gasket is facing bottle**

Check filter to make sure it is pushed into the filter holder (the filter should not come out easily but will move from side to side. There is no locking sound when pushing the filter in).

Check the filter to see if there is **milk in the filter**
Change the filter if there is milk in filter.

PJ's *comfort*®

Quick reference guide for getting started

Joan Ortiz, RN, BSN, IBCLC
available 24/7 for questions
877-546-3742 ext 102
Cell 818-970-9547



Limerick Inc.
the more comfortable approach to breastfeeding

www.limerickinc.com 



Recommended settings for 1st time user

- **Setting for Cycle** turn knob all the way over to the **right** to the **start dot**.
- **Setting for Vacuum** turn knob to the **left** on the **start dot**.



Adjust vacuum only

- Pump for the first minute or two before adjusting the vacuum. Recommend increasing the vacuum by turning the knob to the **right** up to the level that feels comfortable to you.
- Recommended pumping time is about 15 - 20 minutes



Gasket

- Use your finger to run along gasket making sure it is flat against bottle cap.
- The pump will operate best with the **PJ's Comfort®** bottle. Other bottles can be used with the **PJ's Comfort®** **breast pump**.